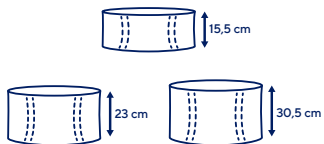
**THUASNE**

# Ceinture abdominale

## Abdominal support

fr	Ceinture abdominale	5
en	Abdominal support	6
de	Bauchgurt	6
nl	Buikgordel	8
it	Cintura addominale	8
es	Faja abdominal	9
pt	Cinta abdominal	10
da	Mavebælte	11
fi	Tukivyö vatsalle	12
sv	Bukbälte	13
el	Ζώνη κοιλιακής χώρας	14
cs	Břišní pás	15
pl	Pas brzuszny	16
lv	Vēdera daļas josta	17
lt	Abdominalinis diržas	18
et	Kõhupiirkonna tugivöö	19
sl	Trebušni pas	20
sk	Brušný podporný pás	21
hu	Hasi támasztó öv	22
bg	Колан за коремната област	22
ro	Centură abdominală	23
ru	Бандаж для брюшного отдела	24
hr	Abdominalni pojas	26
zh	腹部支撐帶	26
ar	حزام للبطن	28

	1	60 - 90 cm
	2	90 - 120 cm
	3	120 - 150 cm

<b>fr</b>	Tour de taille	Hauteurs : 15,5 cm - 23 cm - 30,5 cm
<b>en</b>	Waist measurement	Heights: 15.5 cm - 23 cm - 30.5 cm
<b>de</b>	Taillenumfang	Höhen: 15,5 cm - 23 cm - 30,5 cm
<b>nl</b>	Tailleomvang	Hoogtes: 15,5 cm - 23 cm - 30,5 cm
<b>it</b>	Circonferenza vita	Altezze: 15,5 cm - 23 cm - 30,5 cm
<b>es</b>	Contorno de cintura	Alturas: 15,5 cm - 23 cm - 30,5 cm
<b>pt</b>	Perímetro de cintura	Alturas: 15,5 cm - 23 cm - 30,5 cm
<b>da</b>	Taljemål	Højde: 15,5 cm - 23 cm - 30,5 cm
<b>fi</b>	Vyötäröympärys	Korkeudet: 15,5 cm - 23 cm - 30,5 cm
<b>sv</b>	Midjemått	Höjd: 15,5 cm - 23 cm - 30,5 cm
<b>el</b>	Περίμετρος μέσης	Ύψη: 15,5 εκ. - 23 εκ. - 30,5 εκ.
<b>cs</b>	Obvod pasu	Výšky: 15,5 cm - 23 cm - 30,5 cm
<b>pl</b>	Obwód w talii	Wysokości: 15,5 cm - 23 cm - 30,5 cm
<b>lv</b>	Vidukla apkārtmērs	Augstumi: 15,5 cm - 23 cm - 30,5 cm
<b>lt</b>	Juosmens apimtis	Aukštis: 15,5 cm - 23 cm - 30,5 cm
<b>et</b>	Vööümbmõõt	Kõrgused: 15,5 cm - 23 cm - 30,5 cm
<b>sl</b>	Obseg pasu	Višina: 15,5 cm - 23 cm - 30,5 cm
<b>sk</b>	Obvod pása	Výšky: 15,5 cm - 23 cm - 30,5 cm
<b>hu</b>	Derekbőség	Magasság: 15,5 cm - 23 cm - 30,5 cm
<b>bg</b>	Обиколка на талията	Височини: 15,5 cm - 23 cm - 30,5 cm
<b>ro</b>	Circumferința taliei	Înălțimi: 15,5 cm - 23 cm - 30,5 cm
<b>ru</b>	Обхват талии	Высота: 15,5 см - 23 см - 30,5 см
<b>hr</b>	Opseg struka	Visine: 15,5 cm - 23 cm - 30,5 cm
<b>zh</b>	腰围	高度 : 15.5 cm-23 cm-30.5 cm
<b>ar</b>	محيط الخصر	الارتفاعات: 15.5 سم - 23 سم - 30.5 سم

<b>fr</b>	Compression	Maintien
<b>en</b>	Compression	Support
<b>de</b>	Kompression	Halt
<b>nl</b>	Compressie	Fixatie
<b>it</b>	Compressione	Sostegno
<b>es</b>	Compresión	Sujeción
<b>pt</b>	Compressão	Manutenção
<b>da</b>	Kompression	Støtte
<b>fi</b>	Kompressio	Tuki
<b>sv</b>	Kompression	Stöd
<b>el</b>	Συμπίεση	Συγκράτηση
<b>cs</b>	Komprese	Stabilizace
<b>pl</b>	Kompresja	Podtrzymywanie
<b>lv</b>	Kompresija	Atbalsts
<b>lt</b>	Kompresija	Palaikymas
<b>et</b>	Kompressioon	Hooldus
<b>sl</b>	Kompresija	Oprijem
<b>sk</b>	Kompresia	Podpora
<b>hu</b>	Szorítás	Támász
<b>bg</b>	Компресия	Поддръжка
<b>ro</b>	Compresie	Suținere
<b>ru</b>	Компрессия	Стабилизация
<b>hr</b>	Kompresija	Potpورا
<b>zh</b>	压紧	稳定
<b>ar</b>	الضغط	الدعم

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## ABDOMINAL SUPPORT

### Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### Composition

Elastic fabric.  
Closure with self-fastening surface and thumb-loop.  
Textile components: cotton - polyamide - polyester - Contains natural latex.  
Non-textile components: polyamide

### Properties/Mode of action

The device ensures strong support and adaptable compression to the abdomen and the lumbar area thanks to the elastic fabric.

### Indications

Rib fractures.  
Minor upper and lower back pain.  
Abdominal muscle weakness.  
Postoperative conditions: incisional hernia, umbilical hernia or epigastric hernia, laparotomy, laparoscopy, Laparo-endoscopic single site surgery, cesarean section.  
Postpartum conditions.  
Bruising to the rib cage.

### Contraindications

Do not use in the event of known allergy to any of the components.  
Do not apply the product in direct contact with broken skin.  
Do not use on pregnant women.  
Do not use in the event of a hiatus hernia.  
Do not use in the event of bone cancer with spinal metastases.  
Do not use in the event of circulatory, pulmonary or cardiovascular problems in patients for whom an increase in arterial blood pressure is not recommended.  
Do not use in the event of Pelvic floor deficiency.  
Surgeons usually do not recommend any kind of compression for a period of time after cardiac or thoracic surgery for people at risks.

### Precautions

Verify the product's integrity before every use.  
Do not use the device if it is damaged.  
Choose the appropriate size to fit the patient, referring to the size chart.  
Strictly comply with your healthcare professional's prescription and recommendations for use.  
In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.  
For hygiene and performance reasons, do not re-use the product for another patient.  
It is recommended to adequately tighten the device to achieve support without excessive compression.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

The use of a medical device by a child, should be done under the supervision of an adult or a healthcare professional.

### Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

In postoperative conditions, the restrictive effect of the device could alter respiratory function and decrease lung capacity and tidal volume.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### Instructions for use/Application

Position the belt around the waist, using the thumb-loop.

The bottom of the belt must stop at the top of the crease between the buttocks.

The inside label must be positioned upwards.  
Hold the left end flat using your left hand.  
Pull the thumb-loop simultaneously and evenly.  
Fasten the belt, right side over left side, adjusting to the desired compression.

If the self-fastening strap is properly positioned it does not wrinkle.  
The level of compression can be adjusted as required throughout the day.

### Care

Close the self-fastening tabs before washing.  
Machine washable at 60°C. Do not machine wash the device more than 10 times. Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

### Storage

Store at room temperature, preferably in the original packaging.

### Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2031901 (2021-06)