
THUASNE®

en	Sport knee support	4
fr	Genouillère de maintien sport	5
de	Sport-Kniebandage	6
nl	Steunbrace voor de knie sport	8
it	Ginocchiera di sostegno per uso sportivo	9
es	Rodillera de sujeción para uso deportivo	10
pt	Joelheira de suporte para uso desportivo	11
da	Knæstøttebind sport	13
fi	Polvituki urheilukäyttöön	14
sv	Knästöd sport	15
el	Αθλητική επιγονατιίδα συγκράτησης	17
cs	Kolenní ortéza sport	18
pl	Podtrzymująca orteza kolanowa otwarta sportowa	19
lv	Sporta ceļa atbalsts	21
lt	Palaikomasis kelio įtvargas sportui	22
et	Spordi põlvekaitse	23
sl	Opornica za koleno šport	24
sk	Športová kolenná ortéza	26
hu	Támasztó térdmerevítő sportoláshoz	27
bg	Поддържаща наковенка за спорт	28
ro	Genunchieră de susținere sport	30
hr	Sport steznik za potporu koljena	31

SPORT

Instructions for use - Instructions d'utilisation - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Upute za uporabu

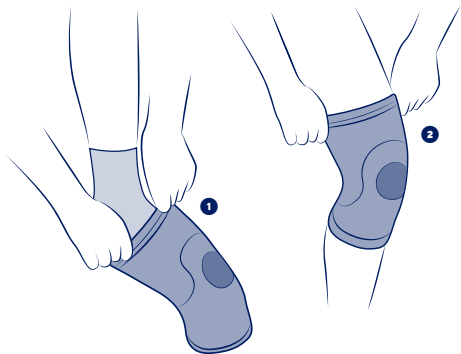cm
S
32 - 34
M
35 - 37
L
38 - 41
XL
42 - 45
XXL
46 - 49

en	Knee circumference in slight flexion
fr	Circonférence de genou en légère flexion
de	Knieumfang bei leichter Beugung
nl	Omvang van de licht geplooidie knie
it	Circonferenza del ginocchio in leggera flessione
es	Circunferencia de la rodilla con ligera flexión
pt	Circunferência do joelho em ligeira flexão
da	Knæets omkreds i let bøjning
fi	Polven ympärys hieman koukistettuna
sv	Omkrets runt knäet i lätt böjning
el	Περιφέρεια γόνατος σε ελαφρά κάμψη
cs	Obvod kolene v mírném ohybu
pl	Obwód kolana przy lekkim zgięciu
lv	Ceļgala apkārtmērs nelielā saliekumā
lt	Šiek tiek sulenкто kelio apimtis
et	Kergelt pinguldatult põlve ümber
sl	Obseg kolena pri rahlem upogibu
sk	Obvod kolena s miernym ohybom
hu	Térd kerülete enyhe hajlításban
bg	Обиколка на коляното при лека флексия
ro	Circumferința genunchiului în flexiune ușoară
hr	Obuhvaća koljeno u blagoj fleksiji

SPORT KNEE SUPPORT

Support level 1

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table. Bilateral product.

Available in 5 sizes.

The device is composed of an anatomically-shaped knitted fabric incorporating a comfort area at the patella and popliteal fossa.

This device can be used when resuming sports activities.

Composition

polyamide - elastane - cotton.

Properties/Mode of action

Compressive elastic fabric ensuring proprioception.

Indications

Knee pain and/or swelling management.

Contraindications

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

For hygiene and performance reasons, do not re-use the product for another patient.

Consult a healthcare professional in case of uncertain diagnosis.

In the event of discomfort, significant discomfort, pain, changes in limb volume, abnormal sensations or color changes in the extremities, remove the device and consult a healthcare professional).

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

In case of doubt before any sports activity, consult a healthcare professional.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated.

Slip the knee brace ❶ onto the leg, making sure that the patella is properly positioned in the flexible knitted area provided for this purpose ❷.

Thuasne Logo must be on the upper part, centered and focused on the patella.

Care/Maintenance

Machine washable at 30 °C (delicate programme). If possible use a washing net. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2048401 (2023-10)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186