
THUASNE®

Ligaflex® Classic

fr	Attelle d'immobilisation du poignet	4
en	Wrist immobilisation splint.....	5
de	Handgelenkorthese zur Immobilisation.....	6
nl	Immobilisatiebrace voor de pols.....	7
it	Tutore di immobilizzazione per il polso	9
es	Férula de inmovilización de la muñeca	10
pt	Tala de imobilização do punho	11
da	Immobiliserende håndledsskinne	12
fi	Immobilisoiva rannetuki.....	14
sv	Immobiliseringsskena för handleden.....	15
el	Νάρθηκας ακινητοποίησης καρπού.....	16
cs	Zápěstní imobilizační ortéza	17
pl	Orteza do unieruchomienia nadgarstka.....	19
lv	Plaukstas locītavas imobilizācijas saite	20
lt	Riešo imobilizavimo įtvaras	21
et	Randme fikseerimise lahas	22
sl	Opornica za imobilizacijo zapetja.....	23
sk	Imobilizačná dlaha na zápästie	25
hu	Immobilizáló rögzítő csuklóhoz.....	26
bg	Шина за имобилизация на китка	27
ro	Atelă de imobilizare a încheieturii mâinii	28
ru	Шина для иммобилизации запястья.....	30
hr	Udlaga za imobilizaciju ručnog zgloba	31
zh	腕部固定夹板	32
ar	جبيرة لتثبيت الرسغ	35

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال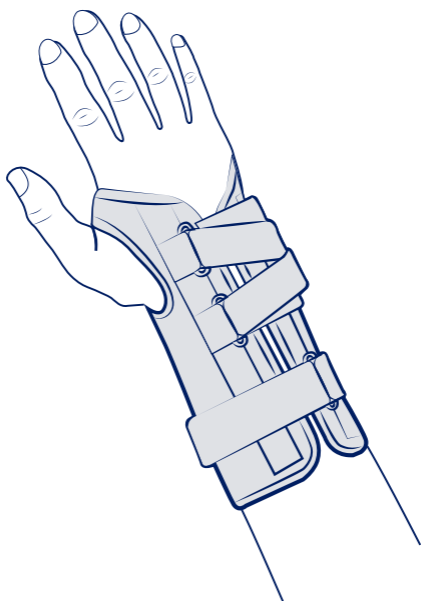

	0	10 - 13 cm
	1	13 - 15 cm
	2	15,5 - 17,5 cm
	3	18 - 20 cm
	4	20,5 - 23 cm

fr	Tour de poignet
en	Wrist circumference
de	Handgelenkumfang
nl	Polsomtrek
it	Circonfrenza del polso
es	Contorno de la muñeca
pt	Perímetro do punho
da	Håndleddets omkreds
fi	Ranteen ympärysmitta
sv	Handledens omkrets
el	Περίμετρος καρπού
cs	Obvod zápěstí
pl	Obwód nadgarstka
lv	Plaukstu locītavas apkārtmērs
lt	Riešo apimtis
et	Randme ümbermõõt
sl	Obseg zapestja
sk	Obvod zápästia
hu	Csukló kerülete
bg	Обиколка на китката
ro	Circumferința încheieturii
ru	Обхват запястья
hr	Obujam ručnog zgloba
zh	腕圍
ar	محيط الرسغ

fr	Immobilisation
en	Immobilization
de	Ruhigstellung
nl	Immobilisatie
it	Immobilizzazione
es	Inmovilización
pt	Imobilização
da	Støtte
fi	Immobilisoi
sv	Immobilisering
el	Ακινητοποίηση
cs	Znehybnění
pl	Unieruchomienie
lv	Imobilizācija
lt	Imobilizavimas
et	Liikumatuks muutmine
sl	Imobilizacija
sk	Imobilizácia
hu	Rögzítés
bg	Обездвижване
ro	Imobilizare
ru	Иммобилизация
hr	Imobilizacija
zh	固定
ar	التثبيت

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WRIST IMMOBILISATION SPLINT

Description/Destination

This device is a wrist splint providing joint immobilisation.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Textile components: polyamide - polyester - cotton - elastane.

Rigid components: polyoxymethylene - aluminum.

Properties/Mode of action

Immobilisation of the wrist via a rigid, malleable and anatomical palmar stay.

Aerated, foam lined fabric.

Dual strap system ensuring even compression.

Indications

Wrist sprain.

Non surgical wrist fracture.

Post-operative or post-traumatic wrist immobilisation.

Wrist tendinopathies.

Conservative treatment of carpal tunnel syndrome.

Lateral epicondylitis.

Rheumatoid arthritis.

Wrist osteoarthritis.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

It is recommended that an adult supervises the application and use of the product by a child.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

If necessary, shape the palmar stay.

Open the splint et position the wrist inside the product.

Adjust the tightening system to close the splint.

Garment care

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label. Remove stay before washing. Replace in the original location before next use. Close the self-fastening tabs before washing. Machine washable at 40°C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2052001 (2023-12)
Thuasne SAS - SIREN/RCS Nanterre 542 091 186